

For use with NJ Phillips 15ml, 20ml, 30ml & 60ml Automatic Drenchers and 25ml, 50ml & 65ml Powered Drenchers.

# Floating Hook

[njphillips.com](http://njphillips.com)

## INSTRUCTIONS

**To ensure satisfactory results, the following procedure must be observed:**

Remove the existing nozzle, nut, valve and spring from your applicator, then connect the adaptor (item 10 or 15) and secure using the nozzle nut (item 11 or 14).

### 20ml, 25ml, 30ml & 50ml Applicators


Adaptor (item 10) is the correct size to be used with the NJ Phillips 20ml Automatic Metal Applicator, 20ml Plastic Automatic Applicator, 25ml Plastic Automatic Applicator, 30ml Metal and Plastic Automatic Applicator and 50ml Powered Applicators.

In all situations when fitting the floating hook the yellow intermediate high-flow delivery valve and spring must be inserted and located between your applicators delivery cage and item 10. If this is not done, the applicator will fail to prime to correctly.

### 60ml & 65ml Applicators

Adaptor (item 15) is the correct size to be used with the NJ Phillips 60ml Metal Automatic Applicator, 60ml Plastic Automatic Applicator and 65ml Powered Applicator.


In all situations when fitting the floating hook leave the delivery valve in the front of your applicator at all times. If this is not done, the applicator will fail to prime to correctly.

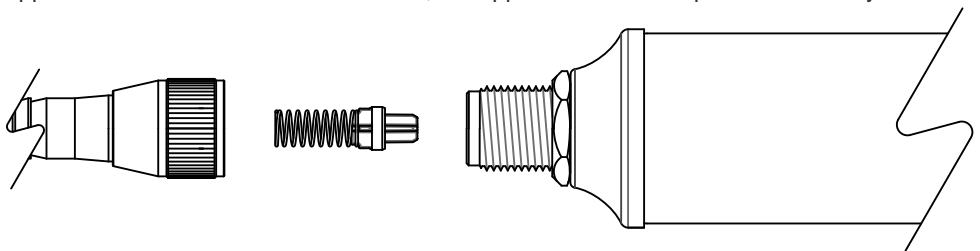

Insert the valve as shown to ensure the applicator primes and sucks fluid into the cylinder.



You must always have the valve and spring (6 and 7) in the front of the hook, otherwise the hook will leak and not prime.

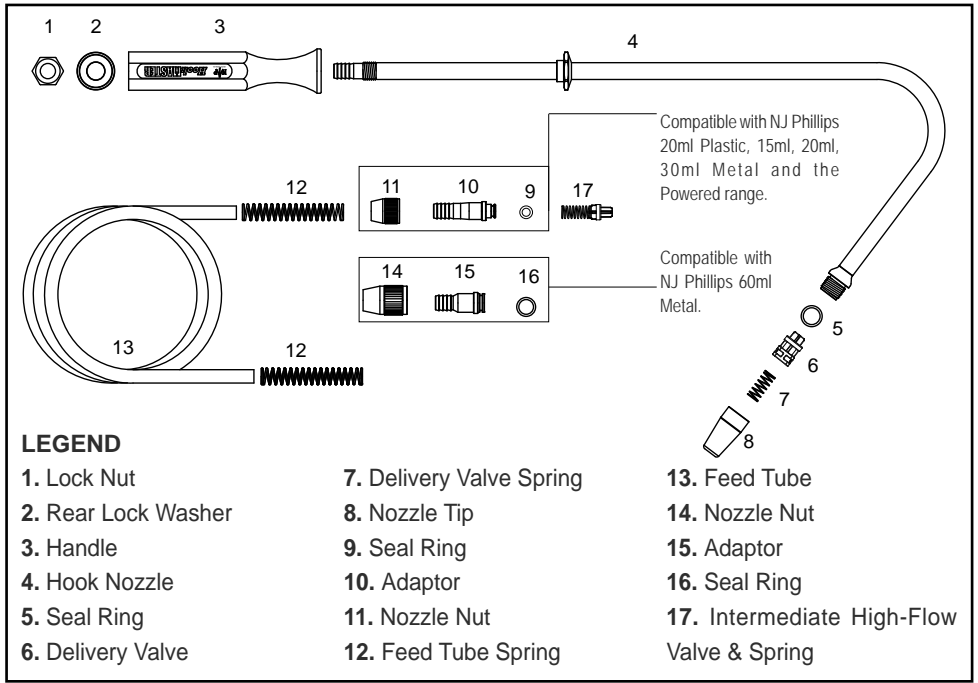

**PLEASE ORDER BY KIT AND PART NAME.**

**SPARE PARTS AVAILABLE:**

<u>KIT No.</u>	<u>PART NAME</u>	<u>INCLUDES ITEM No.</u>
WX1636	Valve Assembly	6,7.
WX1721	Valve Assembly	17.
SN521	Nozzle Tip	8.

AS OUR POLICY IS ONE OF CONTINUOUS IMPROVEMENT THE MANUFACTURER RESERVES THE RIGHT TO ALTER THESE SPECIFICATIONS AT ANY TIME. ALL PRODUCTS PRODUCED BY NJ PHILLIPS PTY LIMITED, ARE IDENTIFIED BY A UNIQUE BATCH NUMBER. THIS IDENTIFICATION NUMBER IS AFFIXED TO THE PRODUCT TO ALLOW TRACEABILITY BY THE MANUFACTURER AND MUST NOT BE REMOVED IF PRODUCT INTEGRITY IS TO BE MAINTAINED.



MADE IN AUSTRALIA

**NJ PHILLIPS Pty Limited** ABN 36 000 082 002

POSTAL ADDRESS: LOCKED BAG 8 GOSFORD NSW 2250 AUSTRALIA  
 TELEPHONE: +61 2 4340 2044 FAX: +61 2 4340 1991  
 EMAIL: njp1@njphillips.com.au



**njphillips.com**

QL646-R3